



CURO-diagnostics ApS

New User Login

11th October 2018

Thank You for buying a CURO from CURO-diagnostics Aps.

Before you can use your CURO you will need to download our free APP from the Apple App Store. Ensure that your iPad has a SIM card and a good strong internet signal. Otherwise ensure that you are in an area with a strong WiFi network that you have access to.

Open the App Store icon and search for "curo equine".

Download this APP to your iPad and then open it.

< Login

Sign Up

Your "curo equine" APP
will open with this page.

Press "SKIP" to continue
with registration.



If you have account ID, enter it or simply skip this form.

Account ID

Password

NEXT

SKIP

Enter your details ..

First name

Last name

Email address

Country

Choose country

City

Street

Postcode

Company

VAT

Password

< Sign Up

Account

When completed,
press "NEXT"

Last name

Email address

Country

Choose country

City

Street

Postcode

Company

VAT

Password

NEXT

Your should now enter
the 7 digit number on the
back of your CURO unit ..
e.g. 1501005, then press
"SIGN UP"



Enter your CURO serial number, please.

SIGN UP

Loading



Welcome


Initializing Data



Your "curo equine" APP will now show this page as it logs you in for the first time.

Whilst initializing, the APP will update any software changes we may have made as well as download the software graphics.

Wait a few seconds for it to finish.

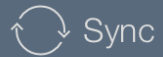

Global search

Horse

Name

Birth year

Sex


Breed

Type

Weight

Workload

Once the "curo equine" APP has initialized you will arrive at this Horse directory page.

Here you can add a Horse to measure, press the plus  symbol in the top right hand corner to get started.



Account



Horses



Denmark

Phone: +45 6146 2732

info@curo-diagnostics.com

www.curo-diagnostics.com